ASSURANCE® INSTALLATION INSTRUCTIONS

3" RAISED PREMIUM TOILET SEAT

888.722.6488 | ToiletSeats.com

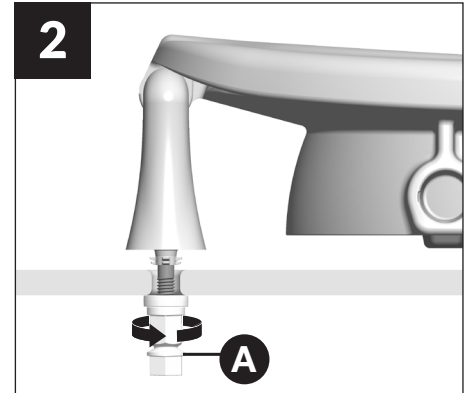
Tool Included
Herramientas incluida
Outil inclus

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


STAY·TITE®

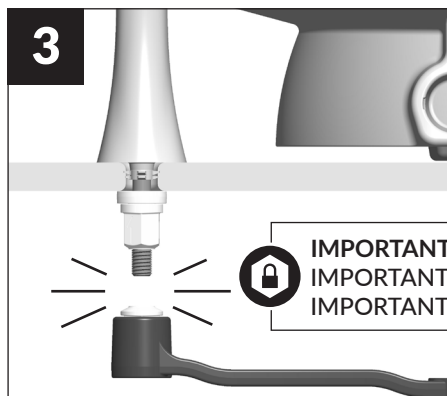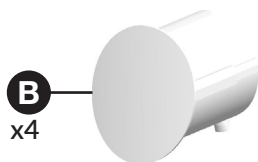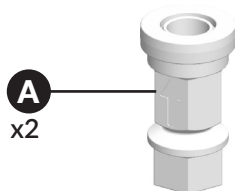
Never Loosens.
Guaranteed.™

3
Holding seat centered on bowl, tighten (A) until lower portions SNAP OFF to prevent over-tightening and will NEVER LOOSEN.

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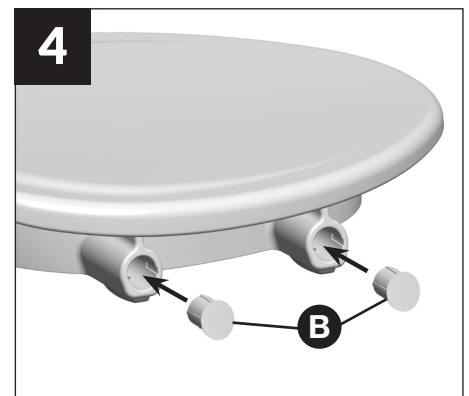4
If you are adding support arms, see installation instructions in support arms package. If not adding support arms, slide caps (B) into holes.

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CARE & CLEANING

Clean only with mild cleaners using a soft, damp cloth, & towel dry.

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LIFETIME WARRANTY

Bemis warrants to the original purchaser that all Bemis Assurance products are free from defects in material and workmanship for the life of the product. This warranty does not cover damage caused by misuse, normal wear and tear or failing to follow manufacturer instructions (including recommended cleaning practices), unauthorized modifications or improper installation.

To submit a warranty claim, please contact your dealer. Please DO NOT return any product to us Bemis without prior consent. If you have questions, please feel free to give us a call!

THIS LIMITED LIFETIME WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. BEMIS' LIABILITY UNDER THIS WARRANTY SHALL NOT EXCEED THE PURCHASE PRICE OF THE PRODUCT.

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